

IMPORTANT: UNPACKING INSTRUCTIONS

AND TABLE RAMP ASSEMBLY

058-800
058-805
058-810
058-815
058-820
058-825



BIODEX

Biodex Medical Systems, Inc.

20 Ramsey Road, Shirley, New York, 11967-4704

Tel: 800-224-6339 (Int'l 631-924-9000).

Fax: 631-924-9241 Email: info@biodex.com, www.biodex.com

This manual covers use of the following products:

058-800
058-805
058-810
058-815
058-820
058-825

NOTE: *At least two people are required to remove the table from the shipping crate.*

Tool Required: Phillips screwdriver

**INSTRUCTIONS FOR UNPACKING TABLE
AND TABLE RAMP ASSEMBLY**

(See Figures 1 and 2.)

The Biodex Table you have purchased is shipped in a crate that contains a ramp as part of the packing material. This ramp can be easily assembled to make it possible to remove the table from the shipping crate without the need for a forklift or other heavy moving equipment.

1. Remove the box from over the table.
2. Carefully remove the shrink wrap from around the table.
3. Using a Phillips screwdriver, unscrew the ramp from the skid.
4. Lift up on the ramp and remove it from the shipping crate.
5. Position the ramp flat on the floor and flush against the front end of the shipping crate, creating a small ramp that leads from the shipping crate to the floor.
6. With the aid of another person, lift up on the table just enough to free the wheels from the cutouts and simply roll the table, slowly and carefully, down the ramp and onto the floor.
7. The table is now ready for installation. Please refer to the Installation and Operation Manual supplied.

NOTE: *If this table is on a trial evaluation you must retain the packing materials in case they are needed to return the table.*

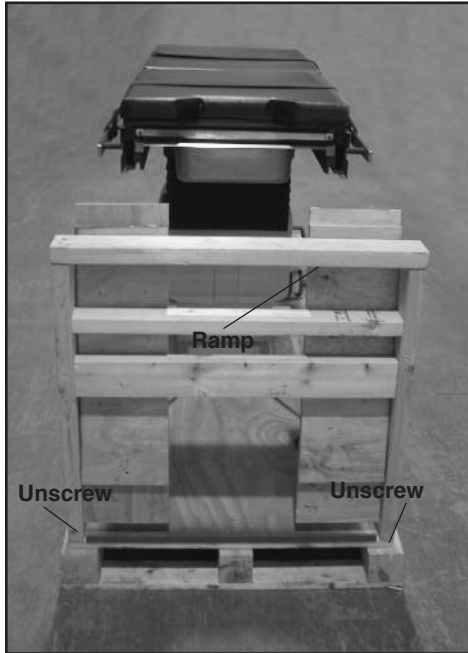


Figure 1: Using a Phillips screwdriver, unscrew the ramp from the skid. Lift up on the ramp to remove it from the shipping crate.

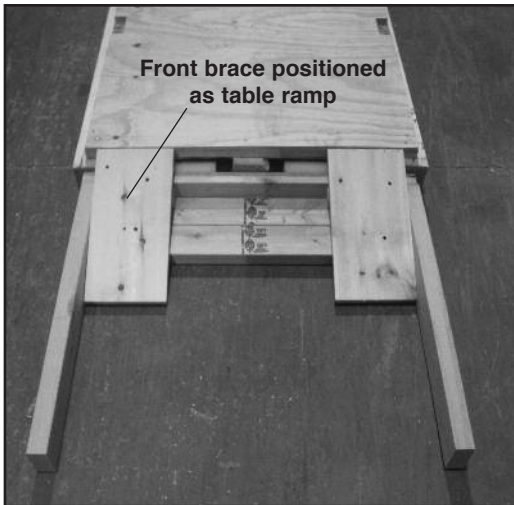


Figure 2: Place the ramp on the floor flush against the front end of the shipping crate to create a ramp. With the aid of another person, lift up on the table to free the wheels and then roll the table slowly down the ramp.

BIODEX

Biodex Medical Systems, Inc.

20 Ramsey Road, Shirley, New York, 11967-4704

Tel: 800-224-6339 (*Int'l 631-924-9000*),

Fax: 631-924-9241 Email: info@biodex.com, www.biodex.com